

DASHTM

IFU



Innovation is our operating tool



STERILE R



Rx only

CE 0483



BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY

DASH™ - Instructions for Use

Caution: Federal (USA) law restrict this device to sale by or on the order of a physician
STERILE. Sterilized by Gamma radiation. For one use only. Do not use open or damaged package
Latex free Radiopaque

Indication of use

DASH™ is a deformable shape memory padded retractor, which is intended to retain or hold back organs and tissue in the course of surgical procedure. At the same time it serves as a means of cushioning the surgical area and protecting organs or vessels from injury.

Warnings

The device shelf life is 3 years from manufacturing date. Do not use device after expiration date.
DASH™ meets the highest standards of safety and reliability. Do not use other than for its intended use.

DO NOT REUSE: DASH™ is designed to be a single use disposable device intended for single patient use only.
Reuse may result in injury and infection.
Cleaning and reuse is strictly prohibited and EZsurgical Ltd can not guarantee device operation if reused.

Storage

Store the device in a cool and dry place.

Instruction for use

DASH™ may be used in laparotomy procedures and procedures within the pelvic area, per the indication of use.

- Before using, it's recommended to wet the DASH™ by saline solution.
- The DASH™ should be shaped and may be reshaped to reach the functional anatomy, and placed within the surgical area in order to lift or retract organs and soft tissues and cushion surrounding tissues or soft organs (such as the intestine or the liver).
- Discard the DASH™ at the end of the procedure to a biohazard waste container.
- DASH™ is X-ray detectable.



World-Wide Distribution Partner



ROBBSURGICAL

510K number: K071775

Robb Surgical Devices, LLC
4547 Chestnut Ridge Road, Suite 110B
Amherst, NY 14228, USA

info@robbsurgical.com
www.robbsurgical.com