



Rx only

**BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY**

DASH™ - Instructions for Use

**Caution: Federal (USA) law restrict this device to sale by or on the order of a physician
STERILE. Sterilized by EtO. For one use only. Do not use open or damaged package
Latex free Radiopaque**

Indication of use

DASH™ is a deformable shape memory padded retractor, which is intended to retain or hold back organs and tissue in the course of surgical procedure. At the same time it serves as a means of cushioning the surgical area and protecting organs or vessels from injury.

Warnings

The **device** shelf life is 2 years from manufacturing date. Do not use device after expiration date.

DASH™ meets the highest standards of safety and reliability. Do not use other than for its intended use.

DO NOT REUSE: DASH™ is designed to be a single use disposable device intended for single patient use only.

Reuse may result in injury and infection.

Cleaning and reuse is strictly prohibited and EZsurgical Ltd cannot guarantee device operation if reused.

Storage

Store the device in a cool and dry place.

Instruction for use

DASH™ may be used in laparotomy procedures and procedures within the pelvic area, per the indication of use.

- Before using, it's recommended to wet the **DASH™** by saline solution.
- The **DASH™** should be shaped and may be reshaped to reach the functional anatomy, and placed within the surgical area in order to lift or retract organs and soft tissues and cushion surrounding tissues or soft organs (such as the intestine or the liver).
- Discard the **DASH™** at the end of the procedure to a biohazard waste container.
- **DASH™** is X-ray detectable.



MedNet GmbH, Borkstraße 10 48163 Münster
Deutschland
Tel: 49-251-32266-0
Fax: 49-251-32266-22
Email: nicole.boehnsich@medneteuropa.com



EZsurgical Ltd., 1 Hadas Street, Or Akiva 30600 Israel
Tel: +972-4-6665060
Email: Info@ezsurgical.com