CardiPatch

ROBBSURGICAL

CardiPatch

Distributed By:

ROBBSURGICAL

640 Ellicott Street (Suite 418)
Buffalo, NY 14203
United States of America
Made in USA
+1 (716) 566-2324

WMDE b.v.
Bergerweg 18
6085 AT Horn
Holland

www.RobbSurgical.com

CardiPatch

(USA) English
CardiPatch sticks to the skin over pacemaker device to help cushion and protect this sensitive area. It contains Silipos® Gel-Care® Advanced, which is a clinically proven gel formula for superior performance in reducing scar color, stiffness, thickness & irregularity. Great for post-surgical wounds!

- Cushions and protects skin around the pacemaker
- Washable and Reusable
- Silicone and Latex Free
- Moisturizes and Conditions while wearing

Directions for Use and Care

- 1.) Wash and dry area thoroughly. Do NOT place product on an open wound
- 2.) Peel off clear plastic.
- 3.) Place gel side directly on skin over pacemaker area.
- 4.) Wear throughout the day. Remove for at least 4 hours daily to allow skin to breathe.
- 5.) Hand wash with soap and water only. Let air-dry with exposed gel facing up. Do not clean with - chemicals.

Warning: If you have diabetes or poor circulation, consult your physician before using this product. If pain, numbness, discoloration, or irritation occurs, discontinue use immediately. If symptoms persist, consult a medical professional. Keep out of reach of children and pets. (USA)

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